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
APPLICATION INSTRUCTIONS

Before applying the LEGEND, ARMOR by DONJOY or 4TITUDE brace, please read these instructions completely and carefully. Correct application is vital to the proper functioning of the brace.


INDICATIONS: • ACL, PCL, ACL/PCL Deficiencies • ACL, PCL, ACL/PCL reconstructions • Collateral ligament deficiencies • Collateral ligament deficiencies with PCL involvement • MCL and/or LCL sprains • Hyperextension

*This product was designed to complement the variety of medical treatments common to the above afflictions.
The frequency and duration of use should be determined by your prescribing Healthcare professional.*

E For **ACL** version, draw the hinges forward slightly before fastening the front tibial (shin) strap (#4). For **PCL** version draw the hinges forward slightly before fastening the front thigh strap (#4). For **CI** version draw the hinges forward slightly before fastening the front tibial (shin) (#5) and the front thigh strap (#6). Do not pull the hinges beyond midline of the leg.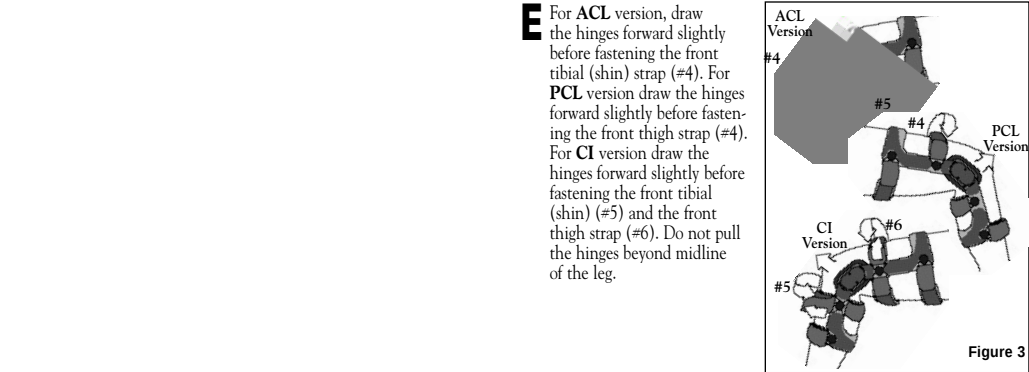



C Keeping the knee bent at 45 degrees, push both condyles (hinges) toward the back of the leg, as shown in figure 2.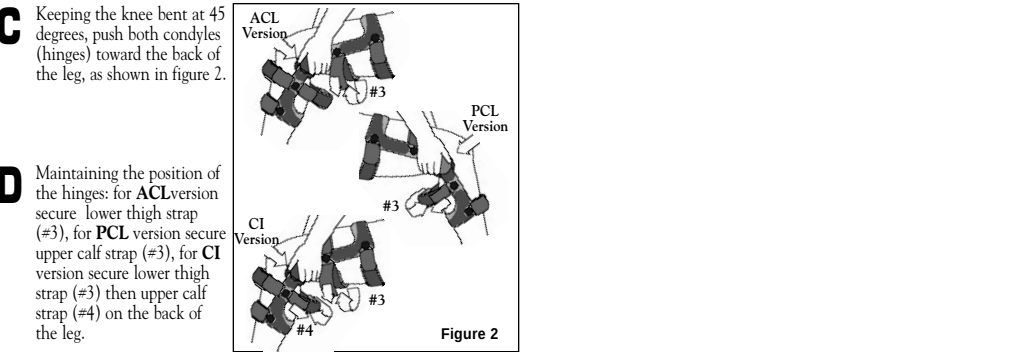

D Maintaining the position of the hinges: for **ACL** version secure lower thigh strap (#3), for **PCL** version secure upper calf strap (#3), for **CI** version secure lower thigh strap (#3) then upper calf strap (#4) on the back of the leg.

INSTRUCTIONS FOR HINGE ADJUSTMENT

Your brace comes with 10° extension stops installed. To change the installed stops, first completely remove the screw as shown (for Armor by DonJoy & Legend figure 5, for 4TITUDE figure 6). Then remove the stops. Insert the new stops, align with the bases on the hinged plate, then reinsert the screw. Using a Phillips head screwdriver, ensure that the screws are seated in the recess and completely tightened. Both medial and lateral stops must be identical in degree.

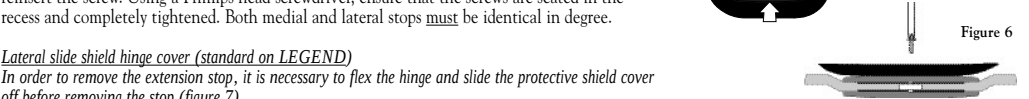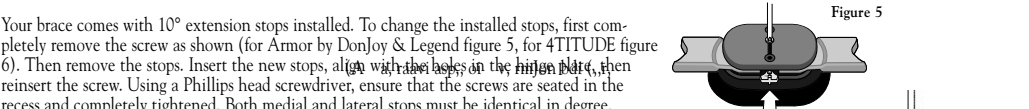
Lateral slide shield hinge cover (standard on LEGEND)

In order to remove the extension stop, it is necessary to flex the hinge and slide the protective shield cover off before removing the stop (figure 7).

The diagram shows a cross-section of the hinge assembly. A black, curved protective shield cover is positioned over a metal hinge mechanism. The hinge consists of a central pivot point with a pin passing through it, and two curved metal plates on either side. The shield cover is designed to protect the hinge mechanism from debris and damage.

Extension Stops: 0°, 10°, 20°, 30°, and 40°.
Flexion Stops: 0°, 45°, 60°, 75°, and 90° (no flexion stops are installed upon shipment)

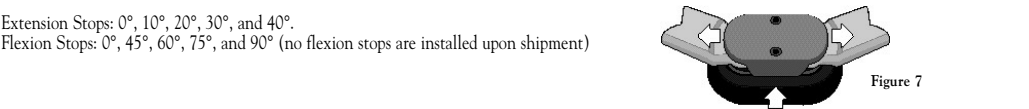USE AND CARE

- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- All straps and pads may be trimmed to fit individual variations in limb circumference.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash your liners and condyle pads in cold water with mild detergent. **AIR DRY ONLY, DO NOT HEAT DRY LINERS OR PADS.**
- The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon™ spray to the internal mechanism of the hinge. Dry lubrication is recommended following use in water.
- Periodically inspect hinge screws for tightness and retighten as needed.

Always consult with your physician or therapist before making changes to the brace.

WARNING: THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: Please be advised that dj Ortnopedics, LLC does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs), during activities (ie . . . skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Functional knee braces are not recommended to be worn over clothing. Consult your licensed Healthcare professional for your activity limitations.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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ATTENTION: LA OI FEDERALS (U.S.A.) RESTREINT LA VENTE DE CE PRODUIT AUX PROFESSIONNELS DE LA SANTÉ.